# Thermo Knife

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# THERMO KNIFE

GB

Thermo knife is an electrical device for waxing up in dental laboratories.

## DESCRIPTION:

It is easy to carry as it is small and very compact, it is also very user friendly with its ergonomic design, convenient and efficient. The thermo knife is equipped with LED lamp of high luminance. The lifespan of the product is long. Precise temperature is discharged as temperature variation is small due to the safe supply of voltage.

## HOLDER:

- User convenience is much enhanced as there is a magnetic holder on the surface of the product.

## CHEST of Drawer:

- TIP can be stored at the chest of drawer equipped at the back side of the product to prevent the loss or damage of the TIP

## Funkcija NAPAKA:

- in case of external error caused by worker's carelessness, a sensor automatically shuts off the circuit to protect the product.

## INSTRUCTIONS FOR USE:

Turn on the power switch after the power cord is properly connected (high brightness LED light will turn on). Mount a carving tip on the electric carving handle and connect the carving handle to the jack.

Set the desired temperature with UP/DOWN key. Temperature is increased or decreased by 1°C every time the UP/DOWN key is pressed one time, and quickly increased or decreased if it's pressed for longer than 1-2 seconds.

Memory function: previous temperature is automatically memorized even when you switch the machine off and turn it on again.

Desired (or frequently used) temperature can be saved at M1 or M2 by following procedures:

- Set desired working temperature
- Press M1 or M2 button for 5 seconds
- Buzzer sound goes off and the set temperature on the screen starts to blink
- The desired working temperature on the screen is saved at M1 or M2

## In case of ERROR

- Buzzer sound goes off and ERROR is displayed on the screen. You have to turn off the device and turn it on again.

## TECHNICAL DATA

Input power source	220-240V/50-60Hz
Fuze	1A
Power taken away at max. temperature	12W
Temperature range	50 – 200°C / 122°F – 392°F
Function signalization:	LED lamp
Outer measures	width=76mm, depth=119mm, height=63mm
Weight:	0,66 kg

## Caution

1. Be careful not to directly touch the holder as it is hot during work.
2. Do not touch the device with wet hands.
3. Precise circuit is built in this product. Excessive power or shock to the product can cause a failure.
4. If there is a defect found during the use of the device do not disassemble it yourself, inquire for service at place of purchase.
5. Turn the unit off and let cool before changing wax tip (there is a danger of burns).
6. Be careful that the tip of the knife does not touch the cable (there is a danger of fire).
7. Do not use damaged power cord, plug or loose outlet (there is a danger of electric shock, fire).
8. Do not bend the power cord excessively or let it be squashed by a heavy object (there is a danger of electric stock, fire).
9. Plug must be connected to the outlet that has an earth terminal (there is a danger of electric stock, fire).
10. Do not block the ventilator located at the bottom of the product (it can cause overheating, fire).
11. Unplug the cord when there is lightning and thunder and when it is not used for a long time (there is a danger of electric shock, fire).
12. Do not pull out the cord or touch the power plug with wet hands (there is a danger of electric shock, fire).
13. Do not install the device where there is much moisture or dust, and the place where water splashes (there is a danger of electric shock, fire).
14. Do not install the device near electric heating appliances (there is a danger of electric shock, fire).
15. Unplug the power cord, and do not directly spray water or wipe it with chemicals etc. (there is a danger of electric shock, fire).
16. Keep power plug prongs free of dust and water (there is a danger of electric shock, fire).
17. If you detect noise or smell smoke coming from the unit, unplug it immediately (there is a danger of electric shock, fire).
18. Do not disassemble, repair, remodel carelessly (there is a danger of electric shock).
19. Repair of the product has to be performed by an authorized service center. Manufacturer will never be responsible for any problems occurred by the misuse of the product in a manner other than prescribed. The warranty of the product is one year and does not apply if the product is used for others purposes as described in these instructions and in the case of tampering with the product at an unauthorized service center.

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*We Interdent d.o.o., Opekarniška cesta 26, SI – 3000 Celje declare that products below are in accordance with EU directives*

**Naziv izdelka: Električni nožek za modeliranje z voskom**  
**Name of the product: electric knife for waxing up**

**Tip: termonož**  
**Type: thermoknife**

**Upoštewane EU smernice:**

**EU directives:**

- EU – nizkonapetostna smernica: 2014/35/EU  
*EU – low voltage directive: 2014/35/EU*
- EU – smernica za elektromagnetno združljivost: 2014/30/EU  
*EU – directive for electromagnetic compatibility: 2014/30/EU*

**Testne metode po standardih:**

**Test methods according standards**

- EN 61326-1:2013
- EN 61000-3-2:2014
- EN 61000-3-3:2013
- EN 61010-1:2010

**Standard kakovosti: ISO 9001: 2015**

**Quality standard: ISO 9001:2015**

Proizvod je varen za uporabo v standardnih pogojih. Vsaka sprememba proizvoda, ki ni odobrena s strani proizvajalca, razveljavlja to izjavo.

*Product is safe in standard conditions. Any change of the product that is not approved by producer, repeals this declaration.*

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- Izdelek bo v garancijskem roku brezhibno deloval, če ga boste uporabljali v skladu z njegovim namenom in priloženimi navodili. / *The product will work flawless during the warranty period if being administered in accordance with its purpose and the instructions provided.*
- Garancijska doba traja 12 mesecev od dneva prodaje. Na vašo zahtevo bomo popravili okvaro in pomanjkljivosti na proizvodu, če boste le-to sporočili v garancijskem roku. Okvaro bomo brezplačno opravili najkasneje v 45 dneh od dneva prijave okvare. Proizvod, ki ga ne bomo popravili v omenjenem roku, bomo na vašo zahtevo zamenjali z novim. Za čas popravila vam bomo podaljšali garancijski rok. / *Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired.*
- Veljavnost garancije dokazujete s potrjenim garancijskim listom in original računom. / *Validity of the warranty should be proofed by certified warranty card and original invoice.*

**Garancija preneha veljati / The warranty expires :**

Če v aparat posega nepooblaščen oseba. / *If an unauthorized person interferes into the device.*

Če niso upoštevana priložena navodila za uporabo. / *If provided instructions for use are not being considered.*

Če je izdelek mehansko poškodovan. / *If the device is being mechanically injured.*

Če so vgrajeni neoriginalni rezervni deli. / *If the unoriginal spare parts are being build in.*

INTERDENT d.o.o.

Opekarniška cesta 26 · SI - 3000 CELJE · SLOVENIA

T: +386 (0)3 425-62-00 · E: [info@interdent.cc](mailto:info@interdent.cc) · [www.interdent.cc](http://www.interdent.cc)

Facebook: Interdent Slovenia