CC DISK Ti5

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EN

Titanium-based casted discs, intended for milling in CAD/CAM machines to produce single crowns, larger bridges and implant superstructures, type 4.

CC DISK Ti5 are made of titanium Ti 6-Al4-V and are used for appliances that have to be rigid and tough. CC DISK Ti5 are free of nickel, beryllium, cadmium and lead and meet the requirements of EN ISO 22674 and EN ISO 9693.

Composition	(Mass- %):	Technical data	
Ti	89,8	Alloy type according EN ISO 22674	4
Al	6	Density	4,43 g/cm ³
V	4	Vicker's hardness	HV 10 353
Fe, O	0,2 % - elongation limit	Rp 0,2	780 MPa (N/mm ²)
	Tensile strenght		860 MPa (N/mm ²)
	Coefficient of thermal expansion	25 - 500 °C	9,8 x 10 ⁻⁶ K ⁻¹
	Ductile yield	A5	10 %

Indications: Used for multiple units fixed dental prostheses for ceramic firing, also for – for appliances with thin sections that are subject to very high forces, e.g. removable partial dentures, clasps, thin veneered single crowns, full arch fixed dental prostheses or those with small cross-sections, bars, attachments, implant retained superstructures.

Contraindications: in case of known allergies to alloy content

Intended users: Product is intended to be used by professionals – dental technicians.

Patient selection criteria: Totally or partially edentulous patients.

Milling: Use suitable milling drills for titan. Smooth the surface of the frame with special cross-cut drills for titan or separating discs, always in same direction to avoid blistering of ceramic. Clean the surface with vapour or diethyleter. Do not use acids like HF.

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Ceramic firing and finishing:

1. Sand blast the surface of the frame under pressure 2-3 bars with 110 μ Al₂O₃ (Interlox).
2. Clean the surface with vapour or diethyleter. Do not use acids like HF.
3. Use ceramic suitable for titanium that meet the demand of the standard EN ISO 9693 (E.G. VITA TITANKERAMIK). Firing has to be done according to the ceramic manufacturer's instructions.
4. On the cervical parts of the crowns, remove the oxides with glass pearls. Use silicone rubbers and polishing pastes for finishing.

Safety precaution! Metal powder is health harmful. Use suction unit and protective msk with filter FFP1.

Side effects: Consider allergic reaction to the alloy composition.

CAUTION! During dry milling of titanium, chips and swarfs can ignite themselves and cause fire. Wet milling is recommended.

Reciprocal Actions! In case of occlusal or approximal contact of different alloys electrochemically based reactions may very rarely occur.

Notice: Any serious incident, that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Warranty! Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our product are subject to further development. Therefore alterations in construction and composition are reserved.

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